








Phone calls made easy with Roger™

Guide



All microphone options at a glance

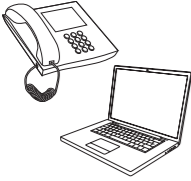

Phone option	Roger On™	Roger Table Mic	Roger Select™	Roger Select™ iN	Page
 PC / laptop for phone calls over internet (via USB cable)	●				3
  Portable device/PC/laptop for phone calls over internet (wireless via Bluetooth®)			●*		4
  PC/laptop for phone calls over internet (via cable)	●	●	●	●	5
 Stationary desktop phone (via cable)	●	●	●	●	6
 Call center (via Bluetooth)			●		7

* This solution does not work with hearing aids with RogerDirect™

The Bluetooth® word mark and logos are registered trademark owned by Bluetooth SIG, Inc. and any use of such marks by Sonova AG is under license.

PC/laptop for phone calls over internet

(VoIP, e.g., Zoom, MS Teams) via USB cable


Required device



Roger On V2
 Roger On iN V2
 Roger On 3

Using USB cable

- 1 Connect the USB cable between your PC/laptop/tablet/smartphone and Roger On
- 2 During the phone call, you can directly speak into your Roger On in three ways:
 - Place Roger On flat on the table
 - Place Roger On tilted on the table so that Roger On points in your direction
 - For best voice pickup, clip Roger On onto your shirt
- 3 Hear the caller's voice in both ears!

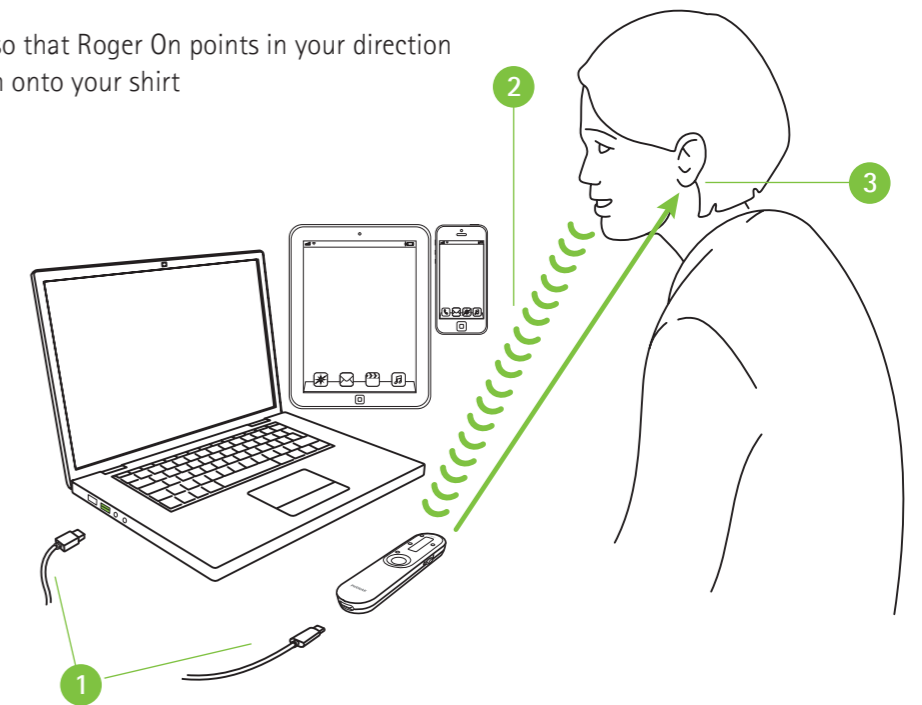

Portable device/PC/laptop wireless via Bluetooth


Required device



Roger Select
Roger Select 3


Using Bluetooth

Simply pair the Roger Select with your Bluetooth enabled portable device/PC/laptop. For making a phone call, you don't need to hold your phone to your ear. And the best thing is, you hear the caller in both ears!

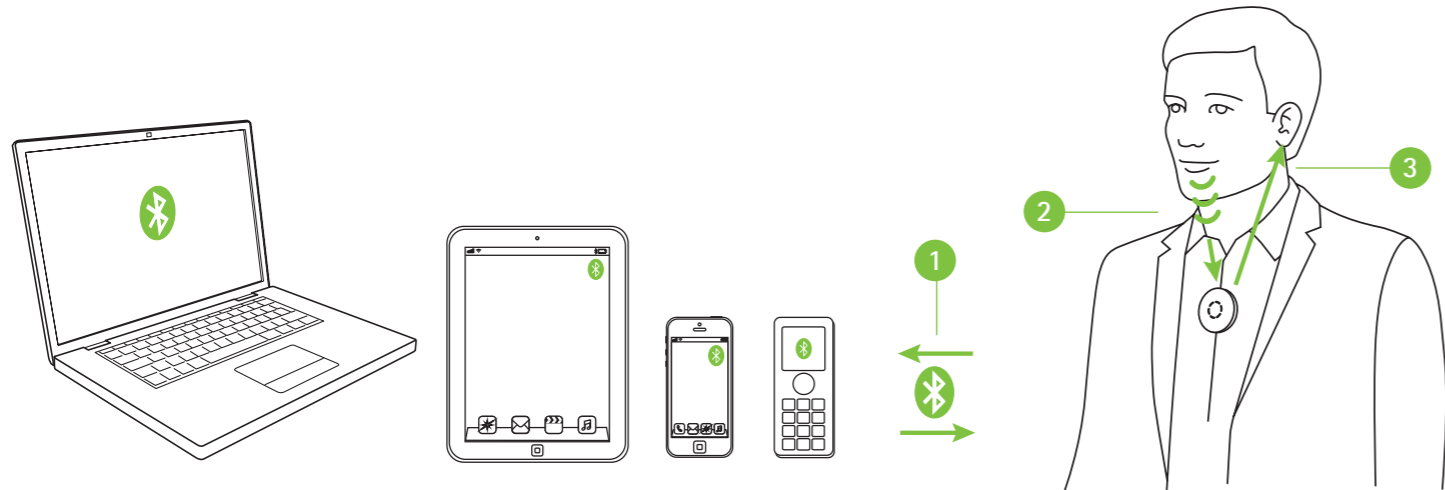

- 1 Pair your portable device/PC/laptop with the Roger Select
- 2 During the phone call, simply talk into the Roger Select
- 3 Hear the caller's voice in both ears!

If your PC/laptop does not have Bluetooth integrated, then plug the Bluetooth USB adapter into a free USB slot on your PC/laptop and pair the Bluetooth USB adapter with the Roger Select.

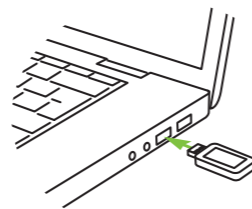

Note: This solution does not work with hearing aids with RogerDirect™.

PC/laptop for phone calls via audio cable

Required devices (options)



Roger Select
all variants



Roger Table Mic
all variants




Roger On
all variants

Using the integrated microphones from a PC/laptop

This easy-to-set-up option sends the caller's voice directly to both ears.

- 1 Connect the headphone output of your PC/laptop to the docking station's audio input or directly to the Roger microphone's audio input
- 2 During the call, speak into the integrated PC/laptop microphone
- 3 Hear the caller's voice in both ears!

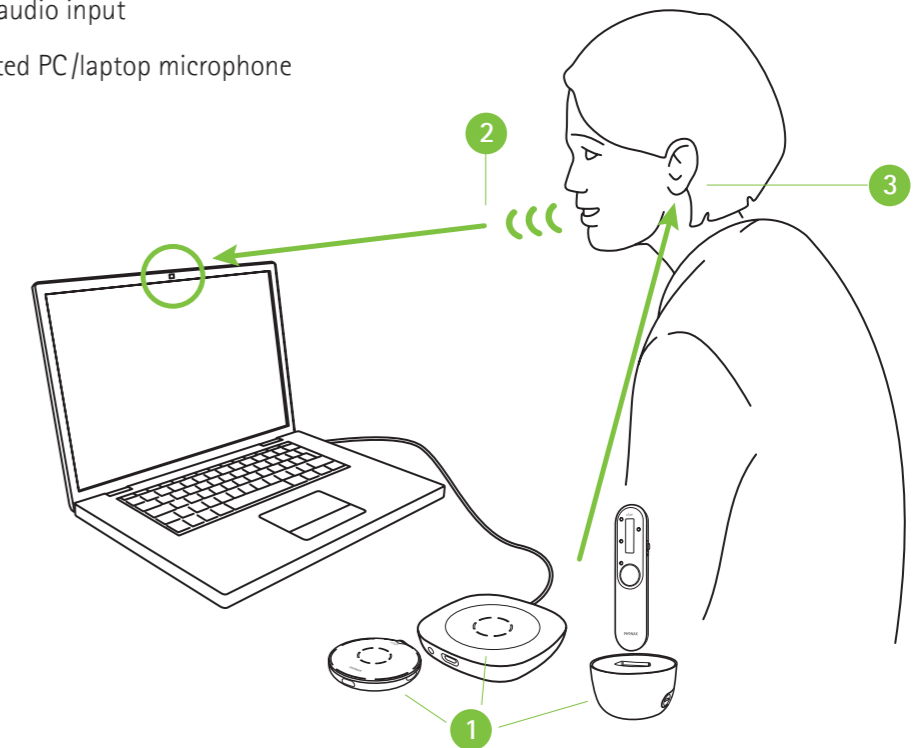

Stationary desktop phones

via cable

Required devices (options)




Roger Select
all variants



Roger Table Mic
all variants




Roger On
all variants

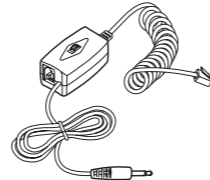

Phone recording adapter
(e.g. Vec LRX-35)

Using phone recording adapter

To answer the phone, pick up the handset and hold it to your mouth because you still need to talk into the handset. The voice from the caller, however, will be sent directly to both ears.

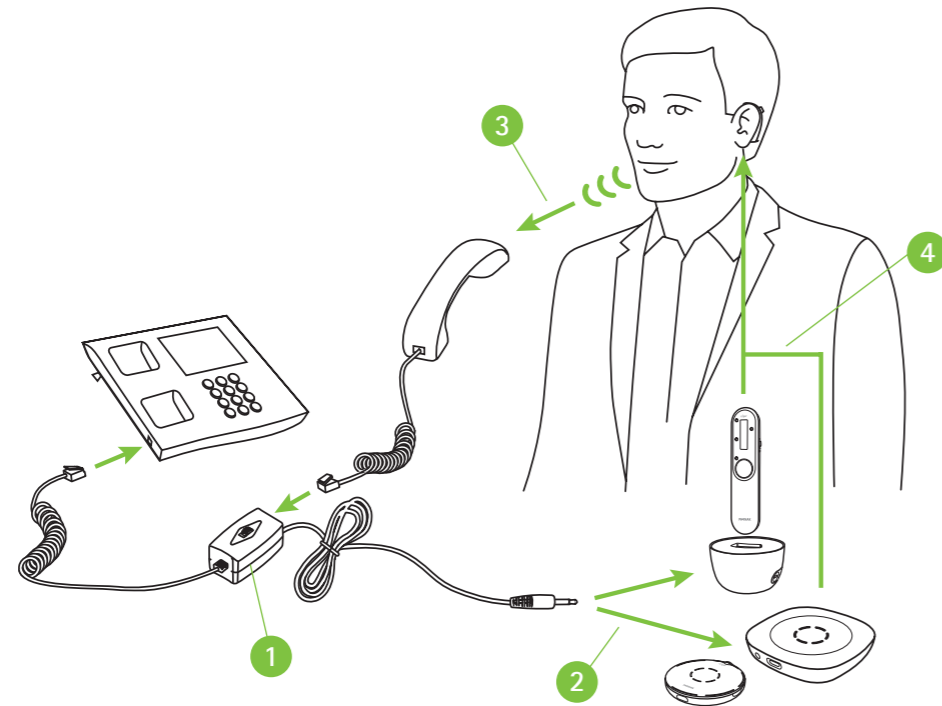

- 1 Install the phone recording adapter between the phone and the handset
- 2 Connect the audio output of the phone recording adapter to the docking station's audio input or directly to the Roger microphone's audio input
- 3 During the call, speak into the phone's handset
- 4 Hear the caller's voice in both ears!



Call center

(PC/laptop and desk phone) wireless via Bluetooth

Required device




Roger Select
Roger Select 3

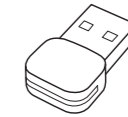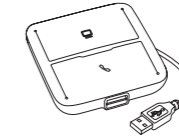

Headset communications hub (e.g., Poly MDA200),
Bluetooth USB adapter (e.g., Poly PLT-BT300)

Using Bluetooth

This is the most advanced solution for busy office calls. Phones can be switched between PC and office phone. In both cases, the phone is transferred to the Roger Select which is ideally worn around the neck enabling you to have both hands free during the phone call.

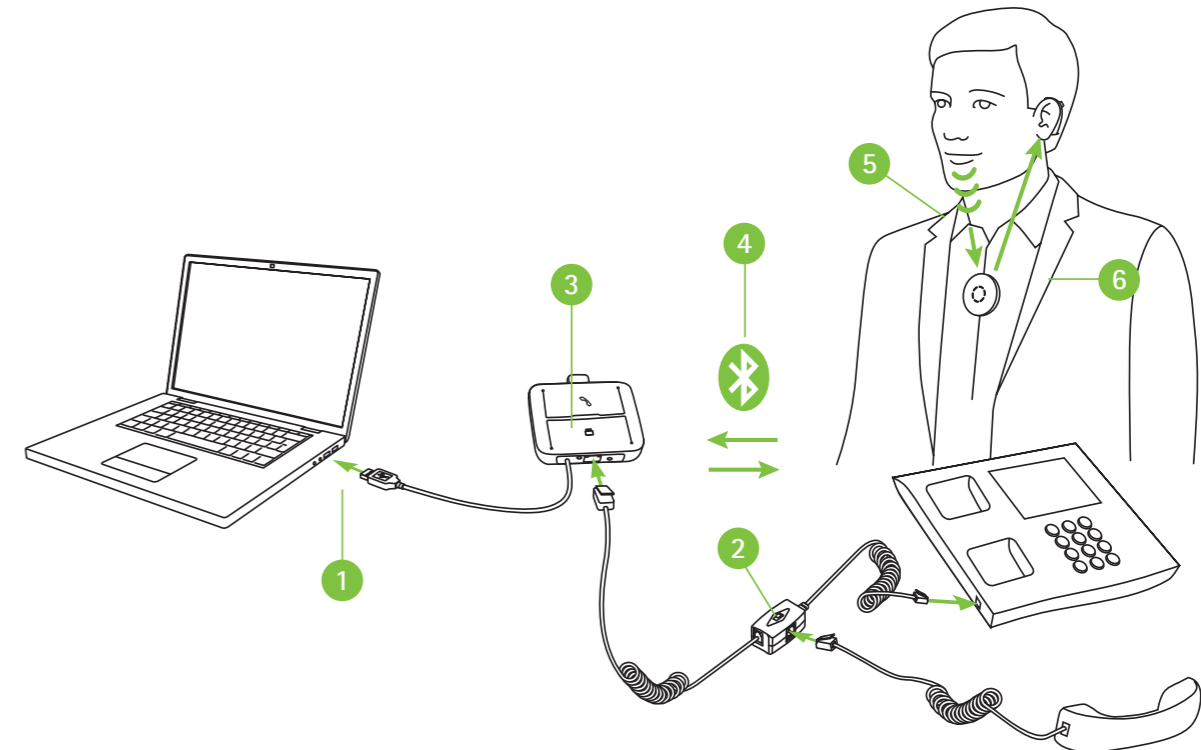

- 1 Connect the headset communications hub to your PC/laptop
- 2 Install the adapter of the headset communications hub between the phone and the handset
- 3 Plug in the Bluetooth USB adapter into the headset communications hub
- 4 Pair the Bluetooth USB adapter with the Roger Select
- 5 During the call, speak into the Roger Select
- 6 Hear the caller's voice in both ears!

Note: This solution does not work with hearing aids with RogerDirect™.

life is on

Since 1947, Phonak is dedicated to preserving life quality by opening new acoustic worlds. Already back then, in the days of our foundation, our company was driven by a formative conviction: We believe that well-hearing equates to well-being and thus is essential for living life to the fullest. In fact, the sense of hearing is directly linked to social, emotional, cognitive and physical well-being. Today as in future, we thrive to offer the broadest portfolio of innovative hearing solutions. And, together with our hearing care professionals, we keep on focusing on what matters most: improving speech understanding, changing people's lives and having a positive effect on society as a whole.

www.phonak.com