

# Phonak Cleaning spray.

The cleaning spray is a cleaning product for the maintenance of hearing aids and earmolds. It removes earwax responsible for the decrease in sound quality, increases the life of hearing systems and preserves their effectiveness.

The cleaning spray is presented in the form of a plastic bottle filled with 1.01 fl. oz (30 mL) of solution. It is equipped with a valve, a diffuser and a brush. The spray button is equipped with an anti-rotation function, allowing a more comfortable use of the spray.

## Properties

The specific formulation of the solution complies with the requirements of the bactericidal standard EN 1040 and the yeasticidal standard EN 1275. The spray meets the requirements of Class I Medical Devices, according to the Regulation 2017/745/EU related to Medical Devices, while respecting environmental and safety requirements for health.

## Using advice

Removal of incrustrated wax:


The cerumen can be fixed on the edge of the outlet openings of the hearing aid as well as nozzle or vents:

- Apply a light pressure to wet the bristles of the brush way from the hearing aid.
- Scrub with the brush to dissolve and remove earwax.
- Wipe your hearing aid with a dry cloth.

Cleaning the surface of the hearing aid:

- Spray the solution on a dry, soft cloth.
- Rub the surfaces of the hearing aid with the cloth.
- After cleaning, wipe the hearing aid with a soft dry cloth.

Be careful not to spray directly on the hearing aid. It is recommended to perform a daily cleaning of hearing aids and earmolds.

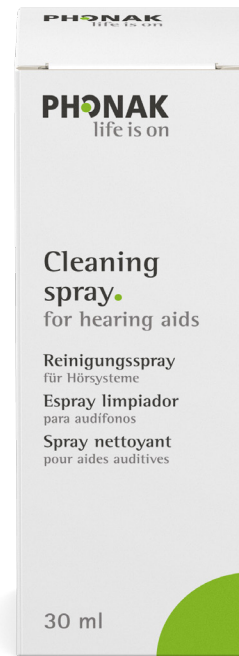

## Product composition

- Water
- Alcohol
- Caprylyl Glycol
- Ethylhexylglycerin
- Benzalkonium Chloride