

Phonak Cleaning tablet.

The cleaning tablet is a cleaning product used to clean the earmolds and removes earwax responsible for the decrease in sound quality, increases the life of the hearing system and preserves their effectiveness. They are presented in packs of 4 cleaning tablets in a 90x90mm scored blister pack and are packed by 5 blister packs (20 tablets).

Properties

The effervescent cleaning tablet composition is a latest technology specially designed to efficiently dissolve the earwax without damaging your earmold. The cleaning tablet meets the requirements of Class I Medical Devices, according to the Regulation 2017/745/EU related to Medical Devices, while respecting environmental and safety requirements for health.

Using advice

- Place the cleaning tablet into the cup then fill two thirds of the cup with warm water.
- Separate the earmold from the hearing aid and drop it into the plastic cup sieve. Never place the hearing instrument in the liquid.
- Leave about 30 minutes.
- Then remove the earmold, rinse it with warm water.
- Dry completely before putting on the hearing aid.


A cleaning tablet is single use.

Discard the solution after each use.

It is recommended to perform a thorough cleaning of the earmold, once a week.

Precautionary statements

- External use
- Single use
- Do not swallow
- Avoid eye contact
- Keep out of reach of children

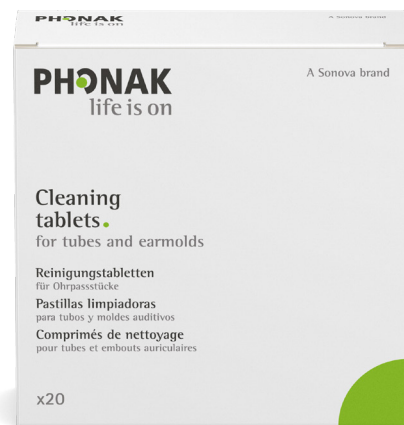

Product composition

- Potassium Monopersulfate
- Sodium Bicarbonate
- Citric Acid
- Sodium carbonate
- Sorbitol