

Phonak Drying capsule.

The drying capsule is indicated for dehumidifying hearing aids and earmolds, in order to avoid breakdown due to moisture.

The drying capsule is a translucent plastic capsule of 30mm in diameter and 12mm in height, which contains crystals of orange silica gel, and closed by a cardboard seal. It's packed by 4, each in an individual sealed bag. Desiccant kits are also available. They are supplied with a specific cup.

Properties

The drying capsule is a maintenance product specially developed to effectively absorb moisture.

The drying capsule meets the requirements of Class I Medical Devices, according to the Regulation 2017/745/EU related to Medical Devices, while respecting environmental and safety requirements for health.

Using advice

- Place the capsule, face upwards, under the foam.
 - Put the hearing aid on the foam with the battery compartment open.
 - Close the cup and let the capsule act all night long.
 - Remember to close the cup after each use.
 - Change the capsule when it becomes almost white.
- The duration of use is about 1 month.

Daily use is recommended, preferably at night.

Regular dehumidification increases the life of hearing systems and preserves their effectiveness.

Precautionary statements

- External use.
- Do not pierce the capsule.
- Do not swallow.
- Keep out of reach of children.



Product composition

- Silica gel