

# Phonak Drying spray.

The drying spray is a care product which facilitates drying of sound tubes and earmolds. It is packaged in 150 or 650ml bottle, provided with a push button.

## Properties

The drying spray has been developed for a quick and efficient maintenance of the sound tubes. The product is totally dry and neutral. It does not damage components or acrylic resins. It meets the requirements of Class I Medical Devices, according to the Regulation 2017/745/EU related to Medical Devices, while respecting environmental and safety requirements for health.

## Using advice

- Use short burst pressure to dry the earmold after cleaning.
- Blow through the hearing tube to remove any water remaining in the tube and earmold.

## Precautionary statements

Extremely flammable spray.  
Contains gas under pressure, may explode if heated.  
Keep out of reach of children.



Product composition  
- Propel 45