

life is on

Since 1947, Phonak is dedicated to preserving social, emotional and physical life quality by opening new acoustic worlds. We believe that well-hearing equates to well-being and is essential to living life to the fullest. Today we offer the broadest portfolio of innovative solutions. And, together with our hearing care professionals, we keep on focusing on what matters most: improving speech understanding, changing people's lives and having a positive effect on society as a whole.

[www.phonak.com](http://www.phonak.com)

**PHONAK**  
life is on

# Phonak cleansing portfolio.



7 461389 1757569

028-2646-02/V1.00/2024-04/km Printed in XXX © 2024 Sonova AG All rights reserved



Sonova AG · Laubisrütstrasse 28  
CH-8712 Stäfa · Schweiz



MG Développement  
Zac les portes de l'aéroport  
178 rue Negue-Cat  
34130 Mauguio-France

**sonova**  
HEAR THE WORLD

# Cleaning

Enhance your hearing aid cleaning maintenance with Phonak's premium solutions



## Cleaning tablets & cup

Utilize the tablets with the cleaning cup to dissolve earwax efficiently, safeguarding the earmold and prolonging the hearing device's lifespan.



## Cleaning wipes & spray

The wipes and cleaning spray are designed to maintain hearing aids and earmolds effectively by removing wax. The wipes are formulated with a specific composition to support maintenance while the spray's anti-rotation function ensures ease of use, optimizing maintenance.

# Drying

Ensure device durability and optimal performance through regular dehumidification



## PerfectDry Lux

The PerfectDry Lux is an electric UV-dryer for the maintenance and storage of all kind of hearing aids. Its 45-minute dehumidifying efficiency has been extensively tested, enhancing the lifespan and effectiveness of hearing devices.

## Drying aerospray

The drying spray ensures quick, efficient maintenance for sound tubes. It's dry, neutral, and won't harm components or acrylic resins.



## Drying kit & capsules

The drying kit is packaged in a case containing a cup and a desiccant capsule. Daily use is recommended, preferably at night.

## Sets

These convenient BTE and ITE kits enable confident and regular at-home maintenance of your hearing aids.

### ITE (in-the-ear) maintenance kit

The kit helps clients to take care and maintain the ITE devices during the daily use. It contains: 10 wipes, 1 cleaning spray (30ml), 2 drying capsules, 1 drying cup.



### BTE (behind-the-ear) starter kit

This essential kit is a must-have to support the daily maintenance of any BTE device. It contains: 10 wipes, 1 cleaning spray (30ml), 2 drying capsules, 1 drying cup, 1 cleaning cup, 1 air blower, 4 cleaning tablets, 1 microfiber cloth.

## Complementary products

All these complimentary items can be purchased separately.



# Essential tips for maintaining your hearing aids

## Preserve dryness

- Ensure your hearing aids stay dry by removing them before showering or swimming
- Keep them clear of moisture-prone areas like bathrooms



## Regular cleaning routine

- Maintain the cleanliness of your hearing aids consistently
- Avoid alcohol, solvents, or other harsh cleaning agents
- Opt for specialized hygiene products recommended by your hearing care professional



## Action

Employ a drying aid nightly. Safeguard your devices with:

- Drying kit: the drying cup and capsule
- Drying spray: drying of sound tubes and earmolds
- PerfectDry Lux: electronic UV-dryer

## Action

Incorporate these cleaning essentials:

- Cleaning spray, cleaning tablets, cleaning wipes, cleaning cup
- ITE maintenance and BTE starter kits will include some of these products

Regular maintenance is key to keeping your hearing aids in great condition. It helps them last longer, avoids breakdowns, and saves you from repairs. Take care of your devices to keep hearing your best!



### Handle with care

- Recognize the value of your hearing aids and handle them delicately with clean hands
- Keep them shielded from dirt, moisture and the reach of children and pets
- Remove your hearing aids before using cosmetics like hairspray, deodorant, perfume, or makeup

